## Eyedrop Guide Dispenser

EN How to use **DROPPY** for bottles:


1. Remove the cap and press the bottle onto the dispenser

2. Pull your skin gently down and place the dispenser to the eye.


3. Close the other eye and focus up towards the light inside the dispenser.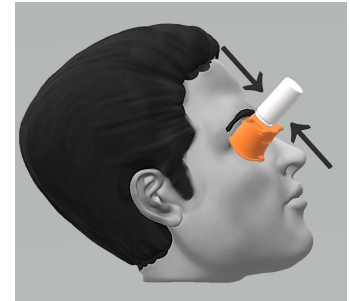


4. Tilt your head backwards and squeeze the bottle lightly.

How to use **DROPPY Clamps** for bottles:

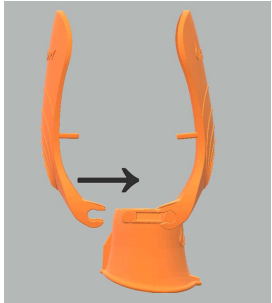
1. Snap the *Clamps* onto the dispenser.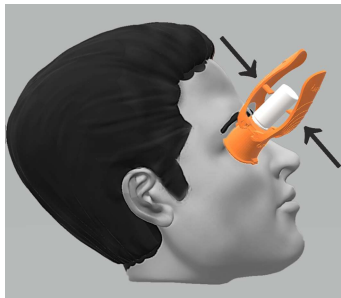

2. Follow the steps as shown above and squeeze the clamps lightly.


How to use **DROPPY Module** for pipettes / vials:

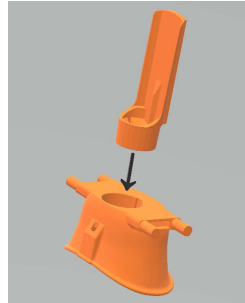1. Push the *Module*

2. Place the pipettes/vials down into the *Module*.

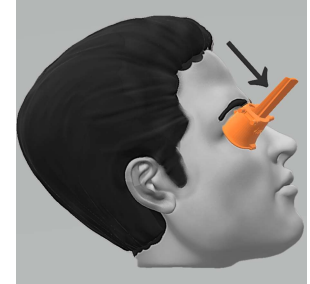3. Tilt your head backwards and squeeze the vial.

- Important:** ○ Always wash your hands before use. ○ Follow at all time the instructions given with the bottle or pipettes/vials that come along with the medicine / lubrication.
- Rinse / wash the dispenser with lukewarm water and dry it with a clean cloth after each time used. To avoid contamination and bacteria growth. ○ Do not clean the dispenser in a dishwasher.
- Do not use the dispenser or related parts if damaged. Recycle the broken product and replace with new dispenser. ○ Make sure that the dropper on the bottles are not too long, to avoid damaging the eye. ○ Keep away from small children. Product is not a toy. ○ Droppy is a single person product. We recommend a separate Droppy dispenser for users of multiple types of eye drops.

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